



# danmeter

Technological Solutions with Human Dimensions

Service Bulletin: CSM060214

Odense 14 Feb. 2006

**Subject:**

CSM Link Software version 3.01

**Issue:**

In CSM Link Software version 3.01 the legends for black and white impedance has been interchanged by mistake.

<b>BS</b> <b>100</b>	<b>EMG</b> <b>0</b>	<b>SQI</b> <b>80</b>	<b>CSI</b> <b>0</b>
Black Impedance 1			
White Impedance 3			
Alarm High Off			
Alarm Low Off			

This picture is as it is shown in CSM Link software version 3.01

The correct reading should be:

**White Impedance 1**  
**Black Impedance 3**

**Cause:**

The texts for black and white impedance has been interchanged by mistake

**Solution/workaround:**

The reading on the CSM monitor itself is correct, so primarily use these readings for impedance check.

**Update:**

This problem will be fixed in a new release of CSM Link Software. (New release not planned yet)

Kind Regards  
Danmeter A/S